



A Used COVID-19 Test Kit

Antigen Test Kit (ATK)

is Infectious Waste

Safely Dispose

- Preventing the spread of COVID-19
- Being responsible for society and environment



1. Spray alcohol or disinfectant all over a used test kit



2. Put it in 2 layers of a bag
It can be disposed together with a used face mask



3. Tie a bag tightly



4. Write "Face mask/ infectious waste" on a bag



5. Dispose separately from regular waste or dispose in a rubbish bin for a used face mask or infectious waste



6. Wash hands with soap abruptly

Cautions

- The antigen test kit must be certified by the Thai Food and Drug Administration (FDA).
- For antigen test, the result will be more precise when testing after contracted 5-14 days.
- The result from an antigen test kit is not specific enough, and cannot be guaranteed according to a virus incubation period. If any symptoms have been developed, conduct the test again for a precision.